

Handle for Q-Flow Fluent and minor lights

Sterilizable detachable handle for Merivaara Q-Flow Fluent, Q-Flow 1 and Q-Flow 2 lights. Can be also used with monitor holders. Recommended focus handle's replacement interval is 1000 sterilization cycles. If the focus handle shows signs of material degradation, replaced immediately.

- The focus handle is removable and separately sterilizable.
- Sterilization of the focus handle can be done in a steam autoclave using the instrument cycle.
- The maximum sterilization temperature for the part in a steam autoclave is 132° C, for three (3) minutes at 2.0 bar. The sterilization time has been calculated by determining when the part being sterilized has reached the sterilization temperature specified above. This does not include the heating and cooling times for the part being sterilized.



Order no.

| | |
|----------|---------------------|
| 485492B | 1 pc |
| 48549210 | Set of 10 pcs, grey |

Technical Specifications

| | |
|--------------|------------------|
| | |
| Product Code | 485492, 48549210 |

Contact us

Merivaara Corp.

Tarmontie 2-4
15860 HOLLOLA
FINLAND

+358 3 3394 611

merivaara@merivaara.com