


## Handle for Intelligent and Vision models

Sterilizable and detachable handle for Merivaara Q-Flow Intelligent and Vision surgical light. Recommended focus handle's replacement interval is 1000 sterilization cycles. If the focus handle shows signs of material degradation, replaced immediately.

- The focus handle is removable and separately sterilizable.
- Sterilization of the focus handle can be done in a steam autoclave using the instrument cycle.
- The maximum sterilization temperature for the part in a steam autoclave is 132° C, for three (3) minutes at 2.0 bar. The sterilization time has been calculated by determining when the part being sterilized has reached the sterilization temperature specified above. This does not include the heating and cooling times for the part being sterilized.

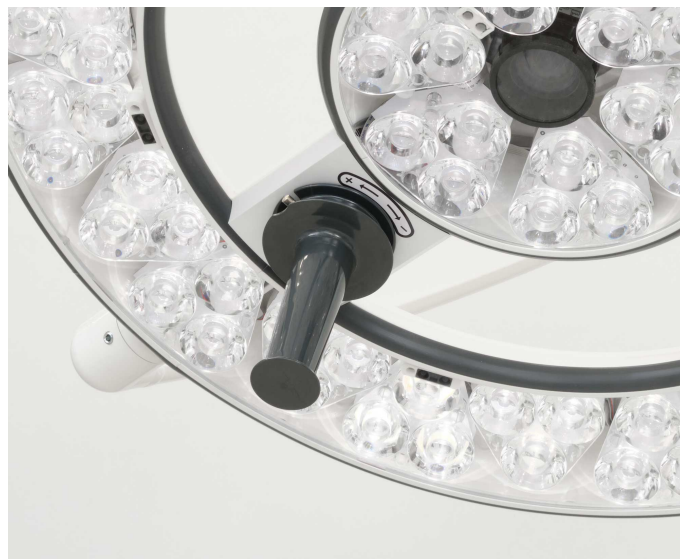

### Order no.

5202801                      1 pc  
A43280901                  Set of 10 pcs, grey

### Technical Specifications

Product Code	5202801, A43280901
Manufacturer	Merivaara Oy

## **Contact us**

**Merivaara Corp.**

Tarmontie 2-4  
15860 HOLLOLA  
FINLAND

+358 3 3394 611

[merivaara@merivaara.com](mailto:merivaara@merivaara.com)